Ambix *activ*

Parenteral pumpe for
mobil bruk

Ambix *activ*

Ambix activ er laget for parenteral ernæring i hjemmemiljø.

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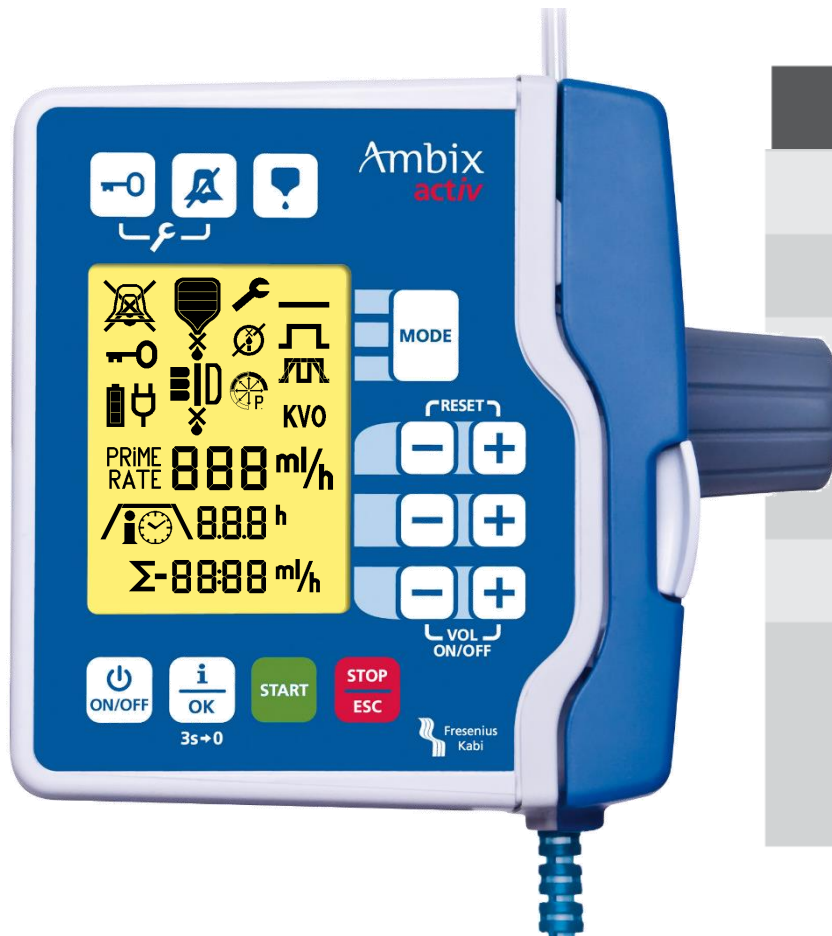
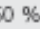
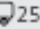
Display symboler

Display Symbol	Meaning	Further details
	Auditory alarm volume level	Indicates the volume of the alarm 🔕 silent, 🔊 low, 🔊 medium, 🔊 high (see parameter setting menu p. 20)
	Keypad locked	Keypad lock is active
	Blinking : Maintenance requested	Pumped volume > 1000 litres
	Keep Vein Open mode	KVO function is activated and target volume selected; if this symbol is flashing, KVO is running (see parameter setting menu p. 19)
	Continuous mode	Continuous mode is active
	Volume/Time mode	Volume/Time mode is active
	Ramp mode	Ramp mode is active
	Active ramping phase	Active phase of the ramping procedure will blink
	Priming mode	Pump is priming
	Battery charging status	charging status of the internal battery pack (🔋 empty battery, 🔋 fully charged battery)

Display symboler

Display Symbol	Meaning	Further details
	Information mode	Mode dependent information will be displayed
	Volume remaining to infuse	Remaining volume will be displayed
	Time threshold	Time thresholds should be programmed
	Pressure indication	Current pressure. If the arrow reaches the black area of the circle, occlusion alarm will be triggered
	Volume remaining to be administered	Volume remaining to be administered  75 %,  50 %,  25 %,  0 %
	Run indicator	The movement of drops and bars indicates that the pump is running
		

FRESENIUS KABI NORGE AS
Gjerdrumsvei 12
0484 Oslo
Tel: +47 22 58 80 00